


Myobrace® for Kids

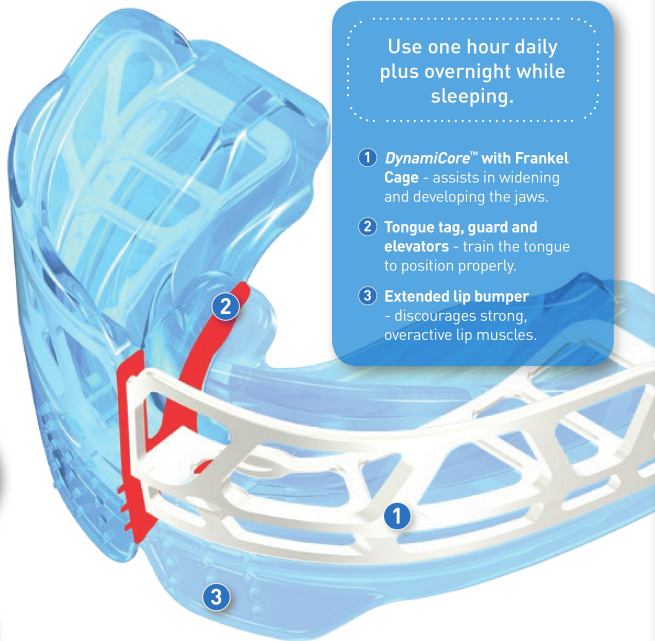

Use one hour daily plus overnight while sleeping.

- 1 **Dynamicores™ with Frankel Cage** - assists in widening and developing the jaws.
- 2 **Tongue tag, guard and elevators** - train the tongue to position properly.
- 3 **Extended lip bumper** - discourages strong, overactive lip muscles.

mixed Dentition

Myobrace® for Kids is a three-stage appliance system designed specifically to correct mouth breathing and poor oral habits, which assists in treating upper and lower jaw development problems. This allows the permanent teeth to align into their natural position. Most effective in the early to mixed dentition stage, 6 – 10 years. *For assistance with appliance selection, use the 'Appliance Selector' feature on myoresearch.com*

Designed for:

- Class II Division 1 + 2 malocclusion.
- Anterior (upper + lower) crowding.
- Deep bite.
- Open bite.



6 - 10 YEARS

All *Myobrace®* appliances are designed to correct habits, develop the maxilla and mandible, and align the teeth. Each stage focuses on a particular treatment goal.



HABIT CORRECTION

Establish nasal breathing.

STAGE 1



4-6 months



The *K1* is available in three sizes and comes in pink, blue and clear.
K1 APPLIANCE - PERSPECTIVE VIEW (left)
K1 - CROSS SECTION (above)

MYOBRACE® K1

The *K1* focuses on establishing nasal breathing and myofunctional habit correction.

It is soft and flexible which gives the best compliance, adapts to any arch-form and malocclusion, and optimises staying in place at night.


Only move to the *K2* when the *K1* stays in at night and nasal breathing is established.




ARCH DEVELOPMENT

Establish correct tongue position.

STAGE 2

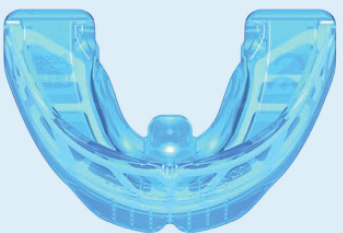

4-6 months

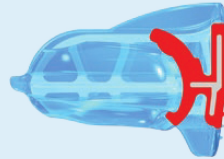

The *K2* is available in three sizes and comes in pink, blue and clear.
K2 APPLIANCE - TOP VIEW (left)
K2 - CROSS SECTION (above)

MYOBRACE® K2


The *K2* focuses on arch development and continuing habit correction. It features a *Dynamicores™* which assists in developing the upper and lower arch form, allowing more room to establish the correct tongue resting position and correct swallowing patterns. Only move to the *K3* when the arch form is improved, correct tongue position and swallowing patterns have been established, along with good dental alignment.




FINAL ALIGNMENT AND RETENTION

Maintain correct lip posture and swallow.

STAGE 3

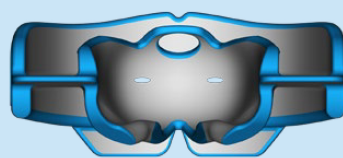

4-6 months

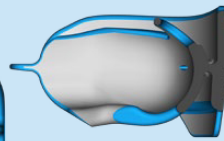

The *K3* is available in three sizes and comes in pink, blue and clear.
K3 APPLIANCE - REAR TECHNICAL VIEW (left)
K3 - CROSS SECTION (above)

MYOBRACE® K3

The *K3* focuses on completing habit correction, final tooth alignment and retention with its firmer polyurethane construction. The hollow tongue tag facilitates final tongue position directly on the correct spot. It also acts as a retainer until the permanent dentition erupts. The *K3* can be replaced with transition to the *T3* and *T4* for final dental alignment in the developing permanent dentition.

Myobrace® for Kids - Broad

Use one hour daily plus overnight while sleeping.

- 1 Patented *DynamiCore™* design - specifically for developing broad arch forms.
- 2 Tongue tag, guard and elevators - train the tongue to position properly.
- 3 Extended lip bumper - discourages strong, overactive lip muscles.
- 4 Patented Broad design - specifically for broader arch forms.

mixed Dentition

The *Myobrace® for Kids - Broad* appliance series is a three-stage system with a specialised arch form for specific genetic and racial groups. This wider arch form is prevalent in many Asian countries, as well as Central and Latin American groups. The appliance is characterised by its flatter anterior arch form with wider canine area, and a more flared posterior arch form. Ultimately, the natural arch form is determined by the tongue position and the *K Broad* series treats breathing, corrects tongue positioning and arch form.

Designed for broad arch forms with:

- Class II Division 1 + 2 malocclusion.
- Class III.
- Anterior [upper + lower] crowding.
- Deep bite.
- Open bite.

6 - 12 YEARS

All *Myobrace®* appliances are designed to correct habits, develop the maxilla and mandible, and align the teeth. Each stage focuses on a particular treatment goal.



HABIT CORRECTION

Establish nasal breathing.

STAGE 1



4-6 months

The *K1 BROAD* is available in medium and large and comes in clear and yellow.

K1 BROAD APPLIANCE - PERSPECTIVE VIEW (left)

K1 BROAD - CROSS SECTION (above)

MYOBRACE® K1 BROAD

The *K1 Broad* focuses on establishing nasal breathing and myofunctional habit correction.

It is soft and flexible which gives the best compliance, adapts to any arch form and malocclusion, and optimises staying in place at night.


Only move to the *K2 Broad* when the *K1 Broad* stays in at night and nasal breathing is established.



ARCH DEVELOPMENT

Establish correct tongue position.

STAGE 2

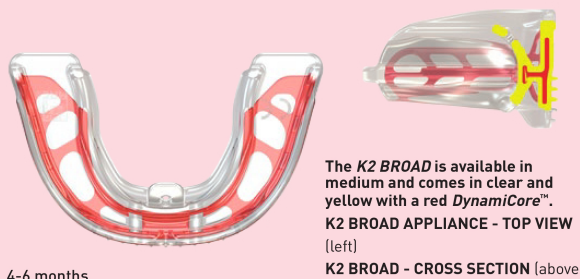

4-6 months

The *K2 BROAD* is available in medium and comes in clear and yellow with a red *DynamiCore™*.

K2 BROAD APPLIANCE - TOP VIEW (left)

K2 BROAD - CROSS SECTION (above)

MYOBRACE® K2 BROAD


The *K2 Broad* focuses on arch development and continuing habit correction. The *K2 Broad* features a *DynamiCore™* which assists in developing the upper and lower arch form. This allows more space to establish correct tongue position and correct swallowing. This, in turn, improves dental alignment. Move to *K3 Broad* when arch form is improved, as well as tongue resting position and correct swallowing patterns are established.



FINAL ALIGNMENT AND RETENTION

Maintain correct lip posture and swallow.

STAGE 3

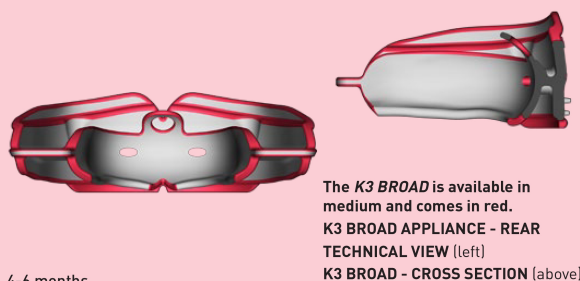

4-6 months

The *K3 BROAD* is available in medium and comes in red.

K3 BROAD APPLIANCE - REAR TECHNICAL VIEW (left)

K3 BROAD - CROSS SECTION (above)

MYOBRACE® K3 BROAD

The *K3 Broad* focuses on completing habit correction, final tooth alignment and retention with its firmer polyurethane construction.

The hollow tongue tag facilitates final tongue position directly on the correct spot.